




Roger™ Touchscreen Mic

Quick guide

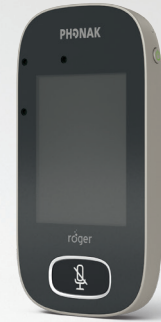

Description

- ① On/Off
- ② Wake/sleep
- ③ Indicator light
- ④ Microphones
- ⑤ Mute
- ⑥ Touchscreen display
- ⑦ micro-USB
- ⑧ Jack 3.5 mm / 1/8"

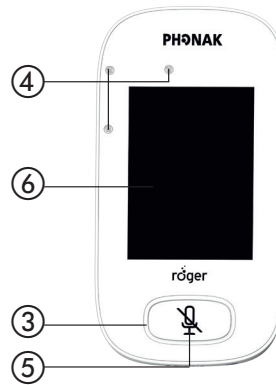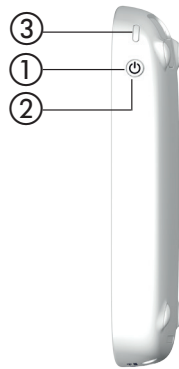

Touchscreen display

	Symbols	Meaning
Always visible		Battery status/Charging/ Low battery
		Remaining use time
		Microphone mode: Lanyard/ Small Group/Pointing
		Network mode: Network/ SubNet
Visible only in use		Audio: Input/Output/Boom
		USB cable connected
		Network role: Stand-alone/ Primary/Secondary

Indicator lights

	Indicator light	Meaning
		On/Off
		Charging
		Fully charged
		Switched on
		Recharge device
		On/Off
		Charging
		Fully charged
		Muted
		Primary microphone not available

Important functions

Connect: Connects receivers/microphones

Settings: Menu for microphone settings

Sub Net: Creates a separate network (private discussion)

New Net: Creates a brand new network

Join Net: Join an existing network

List Devices: Lists receivers/microphones within 4 inches

Check: Provides information about receiver/
microphone within 4 inches

Soundfield*: Adjusts soundfield speaker volume

*Only if you have a DigiMaster connected

Connecting Roger Touchscreen

Press the connect button when you are within 4 inches of the device




Using Roger Touchscreen

1. Lanyard: (Direct instruction)

Teacher wears around neck within 6-8 inches from mouth.


Lanyard

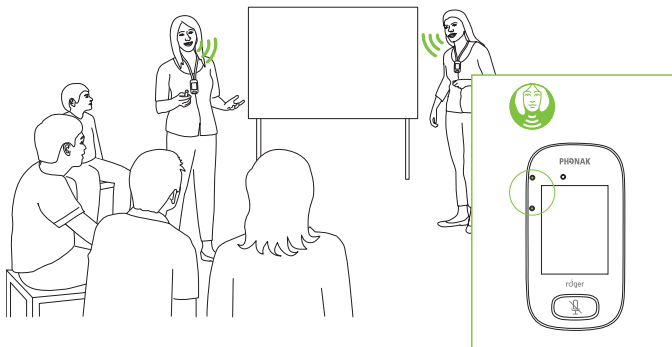

2. Small Group: (Small group work)

Place flat on table to pick up voices of the group.


Small Group

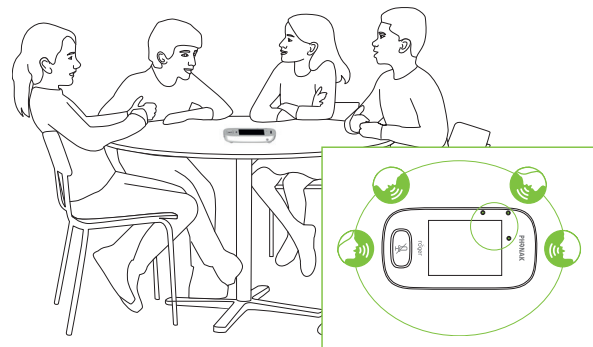

3. Pointing: (handheld use)

Teacher or student can point at the person speaking, within 10 feet away; good for class skits or presentations when multiple people are speaking.

Pointing

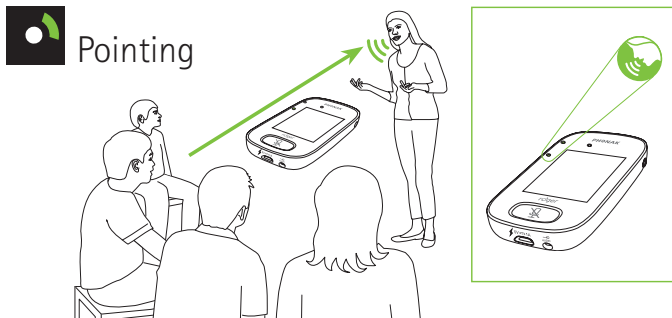

4. Computer/Media Listening:

Plug the auxiliary cable into the touchscreen mic and into computer to replace headphones.

Remember to charge every night.